## Directions For Use

# Visalys Core

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## Product Description

Visalys® Core, developed by Kettenbach, is a dual-curing, fluoride-containing composite used for the fabrication of radiopaque core build-ups and core fillings. The automixing, two-component system is based on a multifunctional acrylic composite.

BPA or BPA precursors are not used in the manufacturing process of this device. Visalys® Core has a firm consistency with good flow behavior and is available in the following shades: white and dentine. Curing can be controlled as required at any time using optional light-curing.

## Indications/Techniques

- Different types of core build-ups
- Luting of root posts.

## Warnings

Do not use in case of a known allergy to methacrylic acid ester or benzoyl peroxide.

Allergic reactions to the product cannot be ruled out with sensitive persons.

Sensitization is possible on contact with the skin/mucosa.

Materials containing eugenol could inhibit the curing of composites.

If necessary, pulp areas of vital teeth should be treated using suitable pulp/dentine protection.

Keep out of reach of children.

## Cautions

Do not use after expiration date.

Do not swallow the material. If swallowed: Consult a medical doctor if problems arise or persist.

Avoid skin and mucosal contact with uncured material.

Avoid contact with the eyes. If there is accidental contact with the eyes: Rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Avoid contact with articles of clothing, as the material cannot be removed by dry cleaning.

Safety data sheets can also be found at [www.kettenbach.com](http://www.kettenbach.com)

## Note

The minimum light intensity should be 800mW/cm<sup>2</sup>. In case of lower light intensity, the light-curing time must be extended accordingly. Place the light as close as possible to the material.

The working times apply for products, which have been stored and processed at a temperature of 23°C/ 74°F and 50% relative humidity. Higher temperatures shorten and lower temperatures extend these times.

## Recommended use

Core build-up

### Complete core build-up:

1. Remove existing fillings and caries.
2. Provide for a dry treatment site using appropriate measures (rubber dam).
3. If the bonding agent requires an additional etching stage, complete etching according to the manufacturer's instructions.
4. Apply the bonding agent according to the manufacturer's instructions.
5. Place a matrix around the tooth if the cavity wall is missing and apply Visalys® Core directly in the cavity. When using a core build-up former, fill it directly using Visalys® Core and place on the tooth. Visalys® Core can be sculpted

during the working time.

6. Visalys® Core cures automatically after 5 minutes. The material can optionally be light cured for 20 seconds to accelerate curing. When using AdheSE or AdheSE One F (Ivoclar Vivadent) and Adper Prompt L-Pop (3M ESPE) adhesive, apply the core build-up material Visalys® Core in incremental layers (2 mm), and light-cure. The curing depth with light-curing is 2 mm.

**Information:** During curing, lower temperatures, e.g. of the product or working in the anterior region, extend the curing time.

7. Remove the matrix or core build-up former, if necessary.
8. Prepare the core after fully curing.
9. Apply a separating medium on the core surface, e.g. using Vaseline, before fabricating a composite temporary restoration.

### Luting root posts:

**Information:** For the luting of root posts it is recommended to use Visalys® Core with the 5-ml-double syringe.

1. Prepare the filled root canal for placing the post.
2. If the bonding agent requires an additional etching stage, complete etching according to the manufacturer's instructions.

3. Prepare the root canal, coronal dentine and, if applicable, the enamel section according to the bonding agent manufacturer's instructions. It is essential to remove any excess from the lumen using a paper tip to prevent blockage of the canal.
4. Condition the surface of the post according to the material manufacturer's instructions. Then avoid contamination of the post surface.
5. Apply Visalys® Core in the canal lumen and, if necessary, on the root post. The tip used should always remain in the material during application. Do not use a lentulo.
6. Insert the root post immediately after application. The working time of Visalys® Core in the root canal is 1 minute.
7. Optional; light cure the material for 20 seconds using a suitable polymerization lamp to fix the post in position.
8. Build up the core as described above.

**Information:** The coronal section of the tooth can be prepared as a core build-up using Visalys® Core. Use of a matrix is recommended during the build-up due to the flowable consistency.

**Application:** The pastes should be mixed and dispensed directly from the 5-ml-double syringe or using an Applyfix® 8 dispensing gun DS24 1:1/2:1 and for the 25-ml-cartridge corresponding mixing tips. Only use the original mixing tips MBT Ø 4.2 mm or MLT Ø 2.5 mm.

The following should be observed when using the 25-ml-cartridge (when using the double syringe, please start at Point 2):

1. Insert the cartridge in the Applyfix® 8 dispensing gun. Ensure that the notches of the cartridge base face downwards. If the cartridge is fitted incorrectly in the dispensing gun, the locking clip will not close.
2. Remove the cartridge/syringe cap and, before fitting a mixing tip for the first time, extrude a small amount of material until both materials are delivered uniformly. Then discard the extruded paste.
3. Fit the mixing tip. Turn the cap 90° in a clockwise direction until it engages. If necessary, fit an endo tip/intraoral tip on the mixer until they engage.
4. Apply constant pressure while extruding material through the mixing tip. The material flow is interrupted as soon as the pressure is eased on the dispenser grip or syringe plunger. Do not use force to extrude material that has already cured in the mixing tips, as the cartridge/syringe and mixing tips could be damaged by creating excess back pressure.
5. Replace the sealing plugs on the cartridge/double syringe.

## Storage conditions

Store dry and protect from sunlight, storage in a refrigerator or at room temperature (4°C / 40°F – 24°C / 75°F), avoid extreme temperatures. The product must be heated to room temperature before use when it has been stored in a refrigerator.

## Cleaning and disinfection of Applyfix® 8

- Remove the cartridge.
- Remove uncured paste using a cloth soaked in alcohol.
- The Applyfix® 8 dispensing gun can be used several times if properly disinfected. Kettenbach recommends exchanging the gun if there are visible traces of use. Disinfection can be carried out using a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions.

## Classification:

ISO 4049, Type 2, Class 3.

## Disposal

Dispose of the product in accordance with local regulations.

## Trademarks

Applyfix® 8 and Visalys® Core are registered trademarks of

Kettenbach GmbH & Co KG.

## Note

Selected Kettenbach materials/products are available in certain markets only.

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

## Limitation of liability

Except where prohibited by law, Kettenbach GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal theory asserted including warranty, contract, negligence or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply and consider all recommendations and information in connection with any use.

Updated: 28th July 2016

## Technical data

Working and curing times:

Working time	90 - 120 sec.
Light-curing	20 sec.
Self-curing	5 min.
Radiopacity	2.0 mm Al
Flexural strength	> 110 MPa

Visalys® Core contains 42 vol% inorganic fillers with a size range of 0.2 - 20 µm.

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