Directions For Use

Visalys® Temp

Gebrauchsinformation	Deutsch..... 2	Instruções de uso	Português..... 53
Directions for use	English 13	Gebruiksaanwijzing	Nederlands 63
Mode d'emploi	Français 23	Информация по использованию	Русский..... 73
Istruzioni per l'uso	Italiano 33	Kullanım Kılavuzu	Türkçe 83
Instrucciones de uso	Español 43	Instrukcja uży	Polski 93

Directions For Use

Visalys[®] Temp

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400 Oser Ave., Suite 1650

Hauppauge, NY 11788, USA

Temporary crown and bridge material

Made in Germany
37289/0813

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

Temporary crown and bridge material

Product description

Visalys® Temp, which was developed by Kettenbach, is a highly esthetic composite for fabricating temporary restorations. The two-component system based on a multifunctional acryl composite is produced using no Bisphenol-A or Bisphenol-A precursors in the manufacturing process. It has similar properties to a filling material and is available in the following shades: A1, A2, A3, A3.5, B1, BL. Visalys® Temp temporary restorations can also be adjusted and customized using flowable light-curing composites.

Indications for use

- Fabrication of temporary prostheses, such as crowns, partial crowns, bridges, inlays, onlays and veneers, for use until a permanent restoration is fabricated.
- Customization of prefabricated acrylic and metal crowns
- Fabrication of long-term temporary restorations

Warnings

Do not use if there is a known allergy to methacrylic acid esters.

Allergic reactions to the product cannot be ruled out with sensitive persons.

Sensitization possible through skin contact.

Visalys® Temp temporary restorations are intended for use as temporary restorations and not as permanent restorations.

Remains of methacrylate-based materials may change the setting properties of silicone impression and bite registration materials. If a restoration has been fabricated previously, prior to impression-taking clean the preparation and surrounding tissue carefully with gauze soaked in alcohol and spray with water.

Temporary luting cements that contain eugenol can inhibit the curing of permanent composite cements, which may be used later for final cementation.

Cautions

Do not use after expiration date.

Do not leave any residue of the temporary crown and bridge material in the sulcus/on the tooth or oral cavity.

Do not swallow the material! If swallowed accidentally: consult a doctor if problems arise or persist.

Avoid contact with the eyes. On accidental eye contact: rinse immediately carefully and thoroughly with eye wash or water. Consult a doctor if problems arise or persist.

Do not inhale grinding dust. Use safety glasses and face mask!

Any area of the impression with severe undercuts and very wide interdental spaces should be blocked out in the usual way.


Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Safety data sheets are available at www.kettenbach.com.

Technical data

Working and setting times

00:00-00:40 min.: sec.	Pouring Visalys® Temp into the impression and placement in the mouth
	Curing in the mouth
01:20-02:20 min.: sec.	Removal from the mouth
	Full curing
From 04:00 min.: sec.	Preparation of the temporary restoration

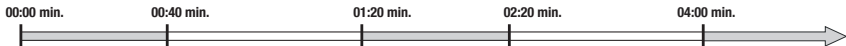Note

Working times apply for products, which have been stored and processed at a temperature of 23°C/ 74°F and a relative humidity of 50%. Higher temperatures shorten the working time and lower temperatures extend the working time.

Preliminary impression

All standard impression materials can be used for fabrication of a temporary crown, bridge, inlay or onlay.

Silginat® is ideal for taking a preliminary impression. As an A-silicone it remains dimensionally stable for several weeks. If a temporary restoration requires remaking, the Silginat® impression can be reused.

Instructions for use

Preparation

First, take a preliminary impression of the dentition.

A laboratory fabricated, vacuum-formed splint or, in the case of single crowns, a prefabricated metal or plastic crown can be used instead of an impression for forming the shape.

The impression can be cut out before use to remove any undercuts and increase the stability of the temporary restoration. A channel can be cut into the impression to create a bar-shaped connection between spaced abutment teeth. Denture teeth can also be used as space maintainers in the anterior region before taking the impression.

Dispensing and mixing

Dispense and mix the pastes using an Applyfix® 6 (DS-50 4:1/10:1 dispensing gun) and corresponding mixing tips. Only use the Kettenbach mixing tips included!

Initial dispensing

- Insert the cartridge into the Applyfix® 6 dispensing gun.
- Remove the cartridge seal and do not use it to reseal the cartridge to avoid trapping air bubbles.
- Before fitting a mixing tip for the first time, advance the plunger into the cartridge and extrude a small amount of material until both materials are dispensed uniformly. Then discard the dispensed paste.

Application

To ensure optimal results we recommend using vinyl or latex gloves.

Place the mixing tip on the cartridge, turn the cap clockwise by 90° until it engages.

With the mixing tip in place, the material requires a certain time before flowing out. The material stops flowing immediately once the pressure on the dispenser handle is

reduced. Do not use force to press out any material that has set in the mixing tip, as it may damage the cartridge and mixing tip! Compomer, resin-modified glass ionomers or composite preparation surfaces must be isolated before application of Visalys® Temp, e.g. using Vaseline.

- Extrude an approximately pea-sized amount of material before each use and discard.
- Fill the dried impression, vacuum-formed splint or preformed crown, beginning at the base. Leave the mixing tip immersed in the material during filling to avoid trapping air bubbles.
- The filled mold must be placed in the mouth using light pressure within 40 seconds.
- The material attains an elastic consistency in the mouth 1 min. 20 sec. after the beginning of mixing and should be removed from the mouth together with the impression or vacuum-formed splint by 2 min. 20 sec. after the beginning of mixing.
- Check the condition of the material in the mouth using the excess material, e.g. with a probe, to avoid it curing too much before removal.
- Leave the filled mixing tip on the cartridge after use as a seal.

- Remove the used mixing tip and dispose of it before reusing the cartridge. Fit a new mixing tip and proceed as instructed.

Finishing

- Completely wipe off the inhibition layer after the material has fully cured, e.g. with alcohol.
- Once the material has cured fully, finish the temporary restoration, e.g. using cross-cut burs and polishers.

Cleaning/Disinfection

- If required, Visalys® Temp can be disinfected using standard aqueous disinfectant solutions, e.g. Silosept®.

Customization/ Recontouring

Basically, the use of a bonding agent is not necessary when using a filling composite in combination with Visalys® Temp for customization and adjusting the shape. Please adhere to the respective instructions for use with other products.

Cementation

- Remove any residual separating agent from the prepared tooth before temporary cementation.
- Standard temporary luting materials can be used for cementation.

- Residue of temporary cements that contain eugenol can impair the curing of composite cements used for subsequent permanent luting. If the plan is to use composite cements, the use of a eugenol-free temporary cement is recommended.

After cementation

- Carefully check the sulcus and surrounding areas and remove any remaining residue of the temporary material or cement.

Repair

- Roughen the fracture site and create mechanical retention on the adjoining sections.
- Clean using Orthoskavident® C or alcohol and then air dry.
- Join or add to newly fabricated restorations using Visalys® Temp or a flowable, light-curing filling composite.
- Join or add to existing restorations using Visalys® Temp or the combination of a flowable, light-curing filling composite and a bonding agent.
- Cure Visalys® Temp or composite according to the respective instructions for use.
- Prepare in the usual way after curing.

Storage conditions

Store in a dry place, protected against sunlight and at room temperature (18 °C/ 64 °F – 25 °C /77 °F). Avoid exposure to extreme temperature fluctuations.

Do not store in a refrigerator.

Disposal

Dispose of the product in accordance with local regulations.

Cleaning and disinfection of the Applyfix® 6:

- Remove the cartridge.
- Remove any uncured paste using a cloth soaked in alcohol.
- The Applyfix® 6 is reusable after disinfection, although it should be replaced if visible signs of wear appear. Disinfection can be carried out using a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions.

Trademarks

Applyfix® 6, Visalys® Temp, Orthoskavident® C, Silginat®, and Silosept® are registered trademarks of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials/products are available in certain markets only.

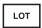
Limitation of liability

Except where prohibited by law, Kettenbach GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

Updated: 18 February 2013

Legends

Conformity with MDD 93/42/EEC		Use by date	
Notified body identification number	0123	Do not re-use	
Temperature limit		Keep dry	
Keep away from sunlight		Catalogue number	REF
Batch code		Caution	
Duales System (Germany only)		Diameter	
Cartridge		Static mixing tip MB (Do not re-use)	
Applyfix® 6 dispensing gun DS 50 4:1/10:1		Rx only For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.	

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