# myCustom resin

powered by: **STYLEITALIANO**

Light cure resin 3 ml

sectional  
matrix  
systems

**Polydentia SA**  
via Cantonale 47 • 6805 Mezzovico-Vira • Switzerland  
**Phone** +41 91 946 29 48 • **Fax** +41 91 946 32 03  
**www.polydentia.ch** • **info@polydentia.ch**  
**Made in Switzerland**

Only for professional use  
REV 03 07.2018

## English - myCustom Resin

### Light cure flowable composite for myCustom Rings kit & Unica anterior introkit

myCustom Resin is a visible light cure flowing microhybrid composite based on tetramethylene dimethacrylate. The blue color facilitates optical control during application.

#### Indication for myCustom Resin

- Shaping of myCustom Rings tines.
- Fixing Unica anterior matrix.

#### Polymerisation

20 seconds with a standard light-curing hand lamp.

#### Storage

Store at 3-25 °C (37-77 °F). Protect from direct sunlight.

Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

#### Additional Notes

- Do not use any resin to adjust viscosity of composite material.
- Contact of resin paste with skin should be avoided, especially by anyone having known constituents allergies.

#### WARRANTY

Polydentia SA warrants this product will be free from defects in material and manufacture. Polydentia makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application other than the indication of the manufacturer.

#### LIMITATION OF LIABILITY

Except where prohibited by law, Polydentia SA will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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